

Needed Tools:
Manufacturers handpiece/coupler wrench

Install Swivel Coupler onto handpiece connector

1. Remove the quick connect coupler from the handpiece. (Fig.1)
2. Insert coupler into handpiece connector, carefully align the coupler to handpiece connector, making sure the rubber gasket touches. (Fig.2)



Fig.1



Fig.2 Insert coupler

3. Firmly tighten the coupler into handpiece nut. Use manufacturers small flat wrench to hold coupler while tightening if necessary.
4. Place handpiece onto the quick connect coupler and test for leaks by running the handpiece.(Fig.3)



Fig.3 Place handpiece onto coupler